## Salvit Respiranto flaster

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## Salvit Respiranto Patch

Patches for improvement of breathing and relief of the discomfort that occurs with nasal obstruction (**Children & Adult**)

- Medical Device Class IIa
- Safety & Scientific Evaluation
- Natural Ingredient
- Effectiveness for 8 Hours
- Night & Sleep Solution for Breath
- Disposable & External Use

### INDICATIONS

Salvit Respiranto patches are used for improvement of breathing and relief of the discomfort that occurs with nasal obstruction. It works by releasing natural vapors of eucalyptus, mint and lavender oil to help clear the nose and breathe better, especially at night time. Salvit Respiranto patches should be applied directly on clothing in the pectoral area, and can be used in children over 30 months (2.5 years) and adults.

### INSTRUCTION FOR USE

1. Open the pouch.
2. Remove release liner and attach the patch directly on clothes (pajamas, etc.) in the pectoral area.
3. Action of natural vapors of the essential oils of Salvit Respiranto patch lasts up to 6 - 8 hours and it is recommended to apply patch before bedtime to cover the period during the nighttime.

### MAIN INGREDIENTS

#### Eucalyptus oil

The mucolytic and expectorants activity of the Eucalyptus oil contributes to the control of hypersecretion in the airway, to secure free breathe.

#### Lavender oil

Inhalation of Lavender oil clears the congested nasal passages and has an antiseptic effect.

#### Dementholised mint oil

It causes effective pain relief and cooling and eases breathing thanks to the improvement of nasal blood. It can considerably reduce the sensation of respiratory discomfort produced by addition of external respiratory loads.

### PRODUCT PRESENTATION

5 patches / box

### SCIENTIFIC REPORT

- Report Procedure: In a Medline search all articles about effects and mechanism of action of essential oils from active ingredient published from 2000 to 2019 were identified.
- Result:
  1. Essential Oils – in general therapy in blocked nose
  2. Techniques of inhalation – application of essential oils to the air to breathe
  3. Eucalyptus Oil, Lavender Oil and Dementholised Mint Oil - application of the each oil in Salvit Respiranto patch to patients with blocked nose that help to refresh the airway.

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